

FM-601

Facial Steamer

User Manual

Welcome to use our products. Rev.A1
Please read the user manual carefully before using the products.

Function description

Functional description: It's a beauty and skincare instrument for cosmetic care.

About This Manual


This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

Features

This series are composed of metal shell , stainless steel base, fashionable glass water cup, copper spray pipe and so forth. Therefore, it is featured with solid structure, durability and good shape. The special stainless steel base guarantees the stability and safety in high temperature, and it will not result in oxidation or re-composition in high temperature, so as to avoid some oxidised and heavy metal atoms from some traditional material in high temperature when they are at work, to harm the skin. Hence, it can guarantee the requirement for cleanness of vapor in beauty treatment. The spray head can rotate at various angles, which make the operation more easily. Apart from the basic herbs function, it is also equipped with time pre-set, five levels spray adjustment, and an automatic water level detection. Besides, it also equipped with maximum and minimum water level alarming system, so as to prevent the damage for heater parts when the water level is too low, or prevent the users from opening the steamer due to their carelessness when there is no water in the water cup.

Functions

1. Intenerate the necrotic skin cells for cleaning in the subsequent treatment.
2. Open up the skin pores for deeper cleaning.
3. Penetrate into pores to soften and clean the grease, black head, make-up residues and dirt.
4. Help skin pores release toxin.
5. Soften wrinkle temporarily
6. Activate the blood circulation, moisten the skin cell, improve metabolism and renew the skin.
7. Help to treat acne, stains, whiten and nourish skin by using appropriate herbs.

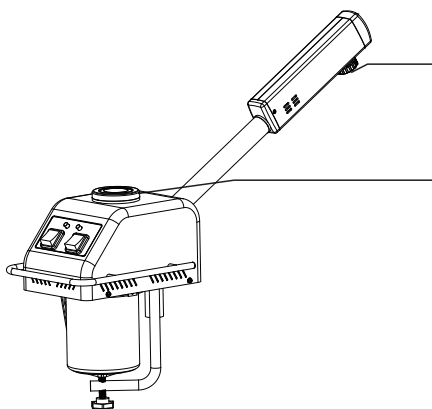


⚠ Warning !

- To prevent from scald, please keep spraying head a certain distance away from face. Spraying head should not point at small area of skin for a long time.
- Also spraying head should not be perpendicular to face. Please stay with appropriate angle.

⚠ Warning !

- Only pure water is available, do not use essential oil and herb.
- Do not remove the cover when machine is heating and becomes steaming within 1mins.

◆ Parts pictures

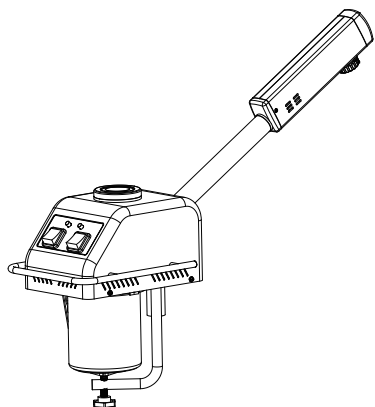


Mainframe



Plastic ring




Herbs cover

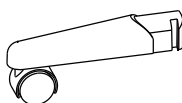

Five-wheel leg × 5



Five-leg base

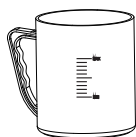

Glass water cup



M14 small panner



Power line



Sliding bar



Adjustment thimble



Pressing plate



Straight post




M10 gasket




M10 screw

Base installation for single function steamer


First mount the five-wheel legs in the five-leg base, and then insert the straight post. (Please refer to pictures No. 1, 2, 3)

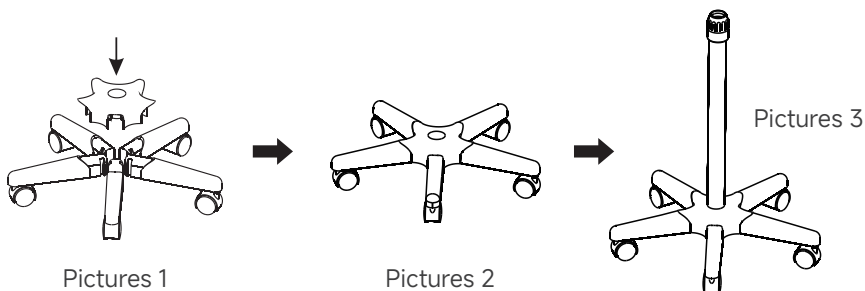

Next, insert the screw which has already set the gasket and pressing plate together to the above-mentioned parts. (Please refer to picture No.4)

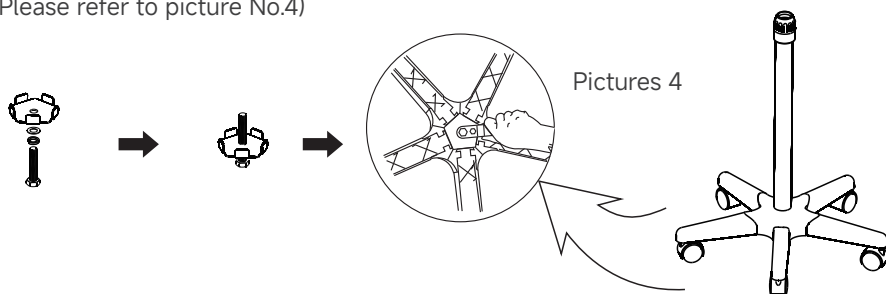

Then, twist the adjustment thimble, insert the sliding bar into the straight post and press it to its lowest position (please refer to pictures No.5 , 6, 7, 9), then adjust it to the suitable height and tighten it. (in picture No.8).

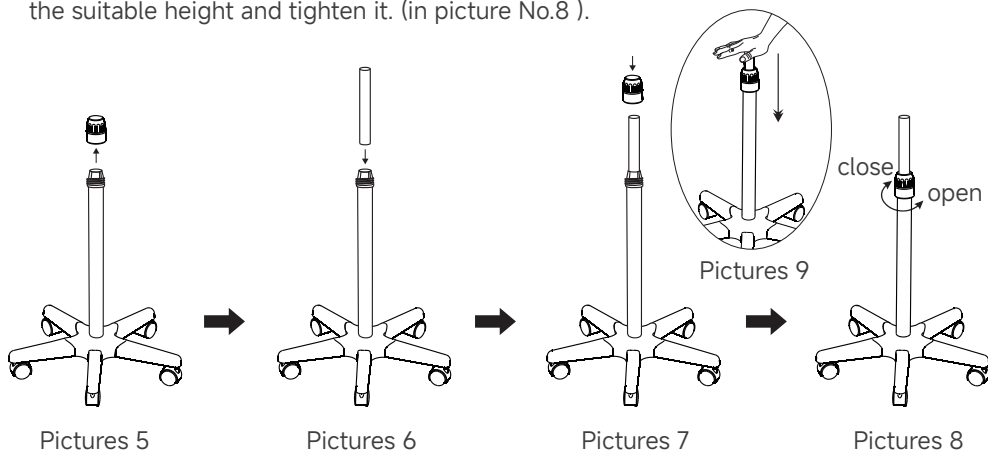


Warning !

When adjust the height of the sliding bar, first you should hold the sliding bar with one hand, then adjust the adjustment thimble with another hand.


Please do not hold the nozzle of the sliding bar to face yourself or others, so as to avoid getting hurt if the sliding bar springs out.

Water cup installation

First, place the plastic ring on the water cup; twist the water cup tray to a suitable place that can hold the water cup and then push it backward; then place the water cup into the mainframe, and push the tray back and then tighten it.
(Please refer to following pictures No. 1 and 2)

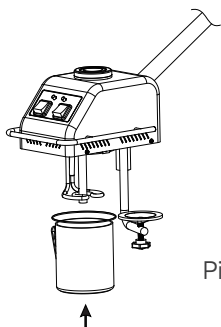

Pictures 1

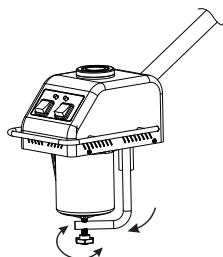


Pictures 2

Warning !


The water cup is designed specially for this series steamer. Please do not use other similar water cups to place into this steamer, for the similar water cups will leak the gas or water easily and cause accidents.

Installation for the whole facial steamer

Please install the steamer onto the sliding bar.
(Please see in the following pictures No.1 and 2)

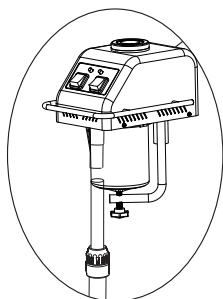


Pictures 1

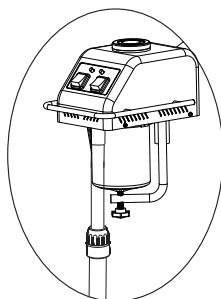

Pictures 2

Operation panel

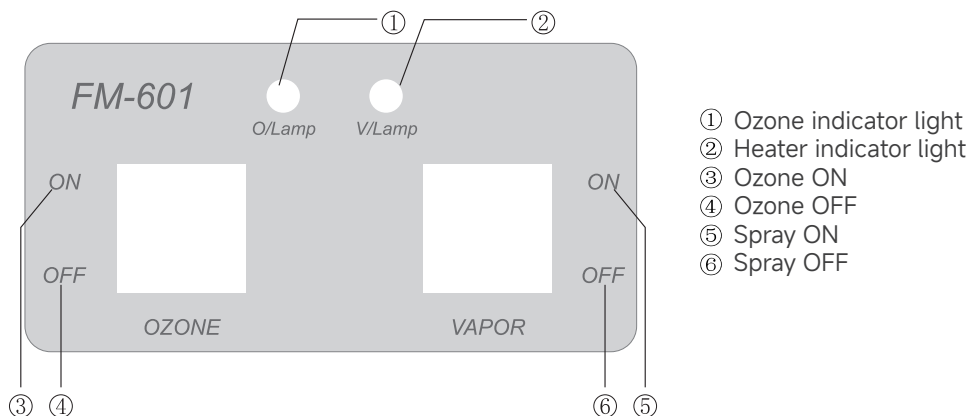

Nursing Preparation


- (1) Adjust the facial chair/massage table to let customer in a stable position and let customer adjust to a comfortable posture.
- (2) Ask customer if there are any contraindications.
- (3) Explain the care contents and precautions to be performed during the operation to customer.
- (4) Beauticians clean and disinfect their hands.
- (5) Help customer clean and refresh his or her skin.
- (6) Test customer's skin: heat & cold sensitivity and skin sensitivity test.
- (7) Place the instrument on a stable place, check the plugs, sockets, wires and connectors to make sure they are safe and keep away from water.

Operation steps

- (1) Open the herbs cover, then pour pure water into the water cup from the water inlet until the water level reaches the highest limit.
- (2) Plug in the power plug.
- (3) Turn on the "VAPOR" (spray switch). When the steam starts to spray evenly and smoothly after starting about 6 minutes, the spraying head can be adjusted to the face be treated.
- (4) Turn on the "OZONE" switch. The indicator light will be lighted up and the ozone tube will be started to work when the power is turned on. It will be accompanied by a distinct smell of ozone when the steam appears. When the steam starts to spray evenly and smoothly, the spraying head can be adjusted to the face be treated.
- (5) While the machine is running, the water level will drop as the steam keeps spraying out continuously. Turn off the power switch and then disconnect the power plug when the water level drops to the lowest level. Add water from the water inlet on the top of the mainframe after about 3-4 minutes. (Refer to the operation step ① but need to be paid attention to the water level between the highest and lowest limits) . Then plug in the power plug to continue working.

◆ Operation method

- (1) Place the water cup upon the water cup tray and then tighten the screw. Next, take the herbs cover lid up and pour the clean water into the water cup. The water level should be between the MAX and MIN limit of the cup. (Please see in the following pictures No.1, 2.)

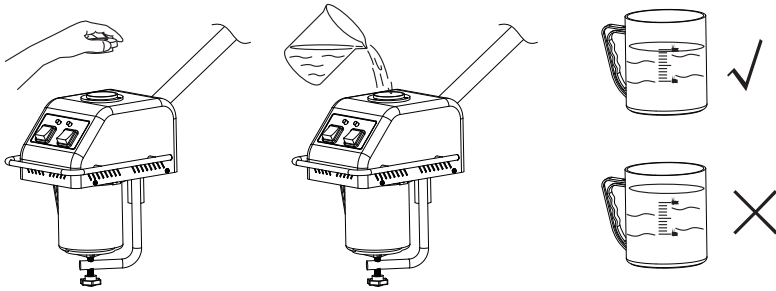

Cautions

- If you need to adjust the height of machine, you should hold tightly the sliding bar with one hand, and twist the adjustment thimble with another hand. Please do not hold the nozzle of the sliding bar to face yourself or others, so as to avoid getting hurt if the sliding bar springs out.
- The protector is used to prevent scald by accidental touching the stainless steel cup while working. Please do not take off the protector. If the protector damages, please return it back to fix.
- The water cup should be placed appropriately. Please pay attention to the position of the rubber ring, which can not be rolled up or folded; otherwise, it will leak due to the deformed plastic ring.
- The water level should be between the "MAX" and "MIN" level limit, when the machine is at work.
- When add water to the water cup, try to avoid spilling water over the surface of the machine shell, otherwise, it will probable cause electric leakage.
- If air or water leakage occurs during working, first switch off the "POWER" button immediately, and cut off power, then take off water cup and check whether the rubber ring is properly fixed, reinstall it. If nothing is improved, please consult the sales agent.
- When operating the machine, nozzle must not aim at client's face until the vapor sprays out thinly and evenly, keep a distance of 40 cm from the face, vapor should spray firstly from client's jaw, then all over the face.
- Since the ozone can kill the skin bacteria, it is not good to use for a long time; otherwise, it will harm the skin and decompose herbs' nutrition. The period of using time should be based on different types of skin.
- During working, when water level falls below the MIN limit, there is no more water into the water cup. It will automatically response immediately and turn off the heater, lock up all the keys except the "VAPOR" key and accompany with alarming by buzzing constant. It won't start to work until the water is added to the normal level, and then repress the key to proceed the operation.
- The interior temperature controller will response automatically, cut off the power for a while. Add some water, otherwise the temperature will drop more slowly.

Since different skin requires different spray distance, the following numbers are as reference

Skin types (remedy suits case)	Distance	Time
Greasy skin, poor blood circulation	35-40cm	About 15 min
Normal skin	35-40cm	10 min
Sensitive skin, dry skin	40-45cm	5 min
Capillary vessel expansion, UV inflammation	40cm	5 min



Cautions

- Forbid to put the greasy stuff into the water container or the herbs filter mesh, otherwise, water may spray out.
- Forbid to spray vapor directly down to the client's face too closely, or it will cause scald.
- Forbid to use the steamer under humid environment, like bathroom, etc.
- Forbid to use the steamer under the sun for long time, otherwise, it will cause some un-stabilities.
- Forbids to conduct electricity between probes directly, or it will lead to elements damages.
- Forbid to open up the machine by oneself. If there is anything unusual, please consult the sales agent, and send for skilled technicians.
- If the electric line was destroyed, you should ask the professional maintain and exchange it avoiding dangerous, or contact the dealer for maintain.
- Please avoid dry operation. Operating without water may shorten the heating element's service life and cause device malfunction.

Maintenance

When operating, please pay attention to the water level of the water cup.

After operation, please clean the product.

1. Fill the kettle with a mixture of white vinegar and water at a ratio of 1:2, ensuring the liquid level covers the heating element / the areas with limescale on the base.
2. Heat to 70-80°C, then turn off the power and allow it to soak:
 - Light limescale: 30-45 minutes
 - Heavy limescale: 1-1.5 hours
 - Stubborn old limescale: 1.5-2 hours
3. Pour out the soaked solution and gently wipe the surface using a soft plastic brush or soft sponge.
4. Boil with purified water to remove any vinegar odor, then rinse thoroughly with tap water or purified water. Check the water cup seal ring every 2 weeks. If it is deformed or deteriorated, please replace it.



CAUTION : Descale Regularly Scale buildup reduces heat dissipation and may cause overheating, element failure, or cracking. Risk increases significantly when combined with dry operation.



Special announcement

- Please make sure the power line is cut off when there is no water or a lack of water. To avoid any danger, forbid to connect the power when the water level is unusual. Otherwise, the users should take the responsibility if there is any accident.
- Please read the detailed instructions seriously or operate the instrument with the help of the professional's technicians.
- Forbid children to use this instrument and get close to or play this instrument within an hour after the instrument stop operating.
- Since this instrument works with high temperature. Forbid nobody to keep an eye on the instrument when it is working to prevent any accidents or fire.

Product Parameters

Net weight	6.5 kg
Gross weight	8 kg
Rated voltage	AC 100-120V / AC 220-240V
Frequency	50/60Hz
Power	650W (AC 100-120V) / 880W (AC 220-240V)

Symbols explanation

	Refer to instruction manual		Manufacturer
	WEEE		CE
	Date of manufacturer		Authorized representative in the European Community
	Warning		Series NO.
	Keep dry from rain rom rain		Fragile, handle with care
	Keep away from sunlight		Stacking limit by mass of 250kg
	Upward		No tumbling
	Be careful		No clasping



The machinery fulfils the following relevant provisions:

Directive 2006/42/EC

Directive 2014/30/EU

Directive 2014/35/EU

Directive 2011/65/EU



Do not dispose of old or defective appliances in domestic garbage;
this should only be done through public collection points.

▲ Product components may be subject to changes without prior notice.
Actual appearance may vary from images.