



Ellora™ Electric Lift Table



Legal Notice

The information provided in this document is intended for informational purposes only and is subject to change without notice. The information provided in this document should not be construed as a commitment by Earthlite®.

Earthlite® assumes no responsibility for the accuracy or suitability of the information contained in this document. This document may contain technical inaccuracies or errors. Earthlite® makes no expressed or implied warranties of any kind concerning the information contained in this document, including, but not limited to, the implied warranties of merchantability and fitness for purpose.

Earthlite® shall not be liable for incidental or consequential damages in connection with or arising out of the product, performance, or use of this document and the information and material it describes.

© Copyright 2021
Earthlite®

All rights reserved. This document or any portion thereof may not be reproduced in any way, including photocopying or translation to another language without the prior written permission from Earthlite.

Earthlite® is a registered trademark of Earthlite, LLC.

INTRODUCTION

Congratulations on your purchase! Earthlite's Ellora™ is perfect for day spas, medi-spas, and resorts. It features a smooth yet powerful electric lift actuator complete with on-demand height adjustments. This stunningly designed table provides exceptional comfort, function, and years of trouble-free use.

The Ellora™ is constructed with state of the art materials beginning with a heavy-duty steel frame. The upholstered tops are available in three (3) separate variations; Salon, Tilt, and Flat. The Ellora™ was designed with operational simplicity in mind. The height adjustment functionality is controlled by a single foot control. The Tilt variation of the table is equipped with electric pneumatic or manual back tilts. The Salon variation of the table is equipped with electric or pneumatic back tilts and leg breaks. There are different mechanisms used on electric, pneumatic, and manual assists.

The following information lists features, specifications, and options for your stationary massage table. There is no assembly required for your Ellora™.

TABLE OF CONTENTS

Introduction	1
Safety, Warnings, and Symbol Definitions	2-3
Specifications	4
Understanding Components	5
Plug and Electrical Information	6
Weight Capacity	6
ETL Compliant Option	7
Accessory Options	7
Removal from Crate	8
Placement & Installation	9
Leveling	9
Operation Instructions	
Important Safety Notes	10
Foot Control	11
Height Adjustment	11
Electric Tilt & Salon Top	11
Manual Back Tilt	12
Pneumatic Back Tilt & Leg Break	12
Chest Comfort System	13
Ellora LX Replaceable Mattress	13
Stretch Assist System	14
Care & Maintenance	
Replacing Foot Glide Leveler	15
Troubleshooting	15
Basic Maintenance	16
Table Model Coverage	16
Lifetime Limited Warranty	17
Contact Us	18

SAFETY INFORMATION

Safety Symbols

Familiarize yourself with the following Safety & Warning symbols. They are designed to prevent damage and injury to you, your clients, and your new Earthlite product.

IMPORTANT: The important safety instructions and warnings in this manual cannot cover all possible problems and conditions that can occur. Use common sense and caution when installing, operating, or maintaining this appliance.



Indicates an imminently hazardous situation which could result in serious or fatal injury.



Indicates a potentially hazardous situation which could result in serious injury.



Indicates a potentially hazardous situation which could result in equipment damage and/or injury.




RISK OF ELECTRIC SHOCK, FIRE, AND EXPLOSION

- 1) Always unplug this furnishing from the electrical outlet before servicing.
- 2) Never use this furnishing if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the furnishing to a service center for examination and repair.
- 3) Keep the cord away from heated surfaces.
- 4) Do not use outdoors.
- 5) Do not operate where aerosol (spray) projects are being used or where oxygen is being administered.
- 6) To disconnect, turn all controls to the off position, then remove plug from outlet.
- 7) Connect this furnishing to a properly grounded outlet only. See Grounding Instructions.

GROUNDING INSTRUCTIONS:

POWERSUPPLY:(120V ACgrounded)Electrical limitations: Do not use extension cord with this product, plug directly into grounded outlet. **This product must be grounded.** If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. This three prong grounded plug will only fit into wall outlet one way. If the plug does not fit fully in the outlet, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

NORTH AMERICA ONLY

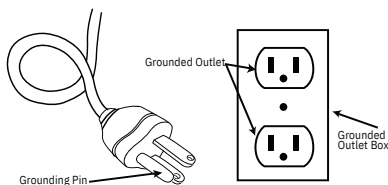

INTERNATIONAL VERSIONS ONLY

This product is for use on a nominal 220V-240V circuit and has a grounding plug (plug type varies by country). Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.

WARNING

To reduce the risk of burns, fire, electric shock, or injury to persons:

- 1) Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
- 2) Do not operate this equipment unless properly trained.
- 3) Install this furnishing in treatment area not accessible to the public.
- 4) Close supervision is necessary when this furnishing is used by, or near children, invalids, or disabled persons.
- 5) This product does not have “pinch point” protection. Only operate when no possibility of any portion of the body could be caught and injured while moving the table top upward/downward, or activating the back/foot tilt.
- 6) Do not use this furnishing to support video equipment such as televisions or computer monitors.
- 7) Never attempt to use the Head or Leg lift section to support concentrated weight such as sitting or standing

CAUTION

- 1) When lowering any section of the table, make sure that no supplies or any part of yours or your client’s body is between the top of your table and its base. This could result in injury, or could damage your table as well as the item. It is recommended that you always return your table top to the lowest flat position to avoid leaving the base exposed.
- 2) When lowering the table height, be sure there are no foreign obstacles are underneath your table that could obstruct it from lowering completely. This could damage your table.
- 3) Never set anything under the table frame to avoid collision if lowered.
- 4) Never operate your table on an uneven surface. Adjust the leveling feet so that your table is stable.
- 5) Never raise the head or leg section with a client in the prone position (face down.)
- 6) Never use or store your table in a wet or very humid environment. Water and excessive moisture can damage the wood cabinet and electrical components. Evidence of exposure to water or moisture will void your warranty.
- 7) If you have purchased the Flexible Armrest Option, never use the Armrests as handholds or body support mechanisms, especially when setting clients on or off the table.
- 8) Never drop or insert any object into any opening.

TABLE SPECIFICATIONS

FEATURES

- Heavy Duty Steel Frame
- Reliable Motor
- Luxuriously Upholstered Top
- Pro-Plush Deluxe 3-Layer 3" Cushioning System
- Hands Free Foot Control
- Comfort-Flex Upholstery - 100% PU in 14 colors
- ADA Compliant
- Lifetime Limited Warranty on Frame, 3 yrs on foam and upholstery, 2 yrs on motor and controls



SPECIFICATIONS

- Widths: 28", 30", 32"
- Length: 73"
- Height Range: 18"-37"
- Lift Capacity: 600 lbs


OPTIONS

- Manual Tilt Top
- Non-Electric Pneumatic-Assist (Tilt or Salon Top)
- Electric Pneumatic-Assist (Tilt or Salon Top)
- Extra Foot Pedal
- UltraLeather, or Promessa Upholstery
- Flex Arms, Neck Roll and Head Pillow (Salon Top Only)
- Flex-Rest or Caress Facecradle
- 4" Memory Foam System
- Face Hole (Non-Tilt End Only)
- Deluxe Adjustable Armrests (Non-Tilt End Only)

Salon

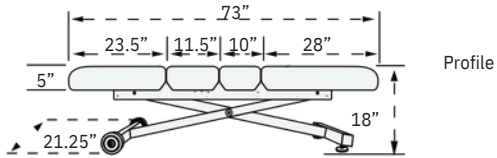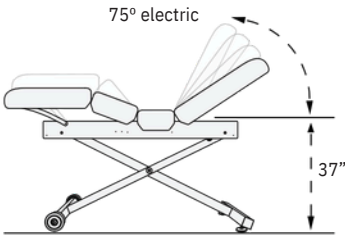


Profile

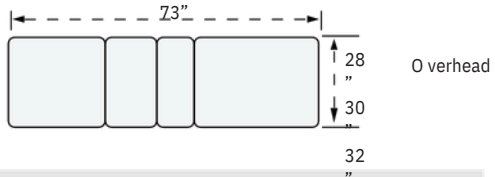


O overhead

Tilt

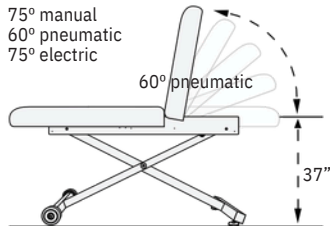


75° manual
60° pneumatic
75° electric

60° pneumatic


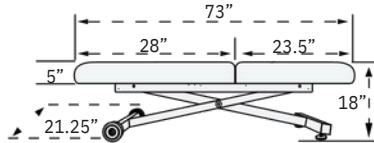


Profile

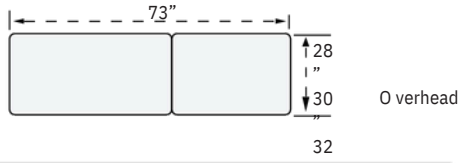


O overhead

Flat

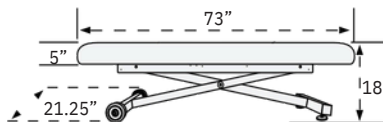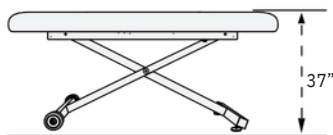

Profile

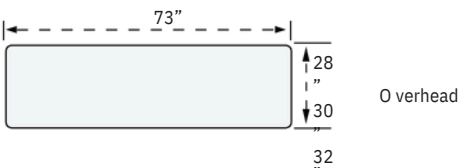

O overhead

UNDERSTANDING COMPONENTS







Getting to Know Your Table

Your new Ellora™ lift table ships ready to connect right out of the box, and knowledge of each common component of the table will ensure a successful experience every time you use it. The following is a list of components that come with each Ellora™ stationary table. Salon and Tilt versions of the Ellora™ include additional components. The following terminologies will also be of helpful assistance should you need to contact us:

COMMON COMPONENTS

- Actuator (1)
- Scissor lift, steel frame (1)
- Upholstered top (1)
- Foot control (1)
- Foot control extension cable (1)
- Back wheels (2)
- Adjustable foot glide levelers (2)
- Maple Headrest Outlets (2)
- Power cable (1)



Common Components

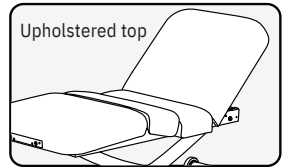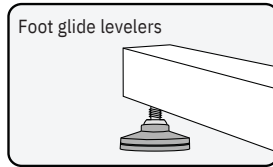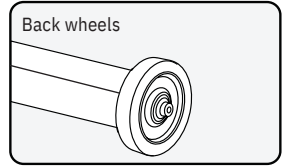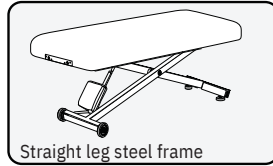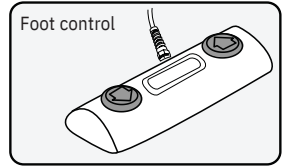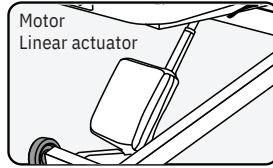

ADDITIONAL COMPONENTS: SALON

- Pneumatic back tilt (1) and leg break (1)
- Electric back tilt (1, leg break (1) and hand control (1)






Additional Components

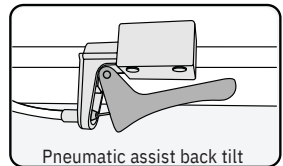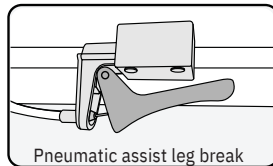





ADDITIONAL COMPONENTS: TILT

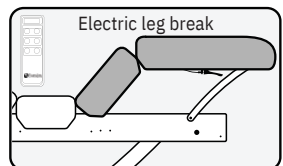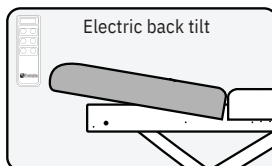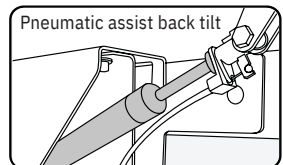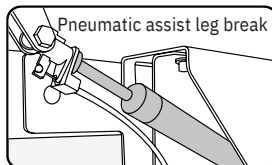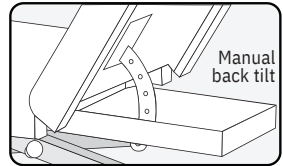- Manual back tilt (1)
- Pneumatic assist back tilt (1)
- Electric back tilt (1) and hand control (1)

Additional Components



PLUG DESCRIPTIONS & ELECTRICAL SPECIFICATIONS

China			
PLUG DESCRIPTION			
North America	UK / Middle East	Asia	Europe
120V AC	220V AC	220V AC	220V AC
			

ELECTRICAL SPECIFICATIONS		
SERVICE	120V / 60Hz 2.5A	220V / 50Hz 1.5A
MAX. AMPERAGE		220V AC
VOLTAGE	120V AC	287.5 WATTS
MAX. CONSUMPTION	287.5 WATTS	


WEIGHT & LIFT CAPACITIES

600 lbs.

The Ellora is powered by a heavy duty, quite lift actuator to enable easy height adjustment, and positioning. The Recommended lift capacity for the Ellora massage table is 600 lbs.

50 lbs.

The back rest and leg section of the table has a maximum weight capacity of 50 lbs.

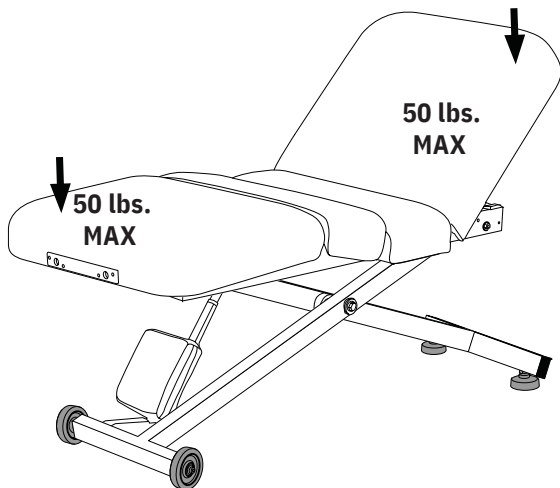

Safety Lockout

Your table may be equipped with a safety lockout. The table will only operate when the magnetic pin is inserted into the lock. Remove pin when table is not in use to prevent unauthorized use and potential injury.



Table will only operate when magnetic pin is inserted



Conforms to
UL STD 962



Intertek
5003418

Certified to
CSA STD C22.2
No. 68

ACCESSORY OPTIONS



Flex-Armrests

The Ellora™ Salon is retrofitted with slots on each side of the table specifically designed to provide a fixed location for Flex-Armrests (**Diagram A**)

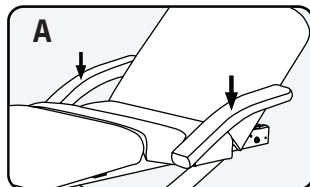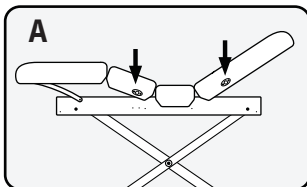

Facecradles & Salon Pillows

All Ellora™ massage tables come with headrest outlets at both ends, specifically designed for providing a fixed location for facecradles and salon pillows (**Diagram B**).

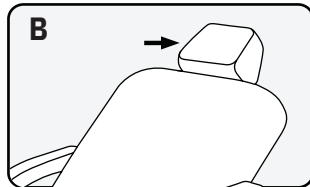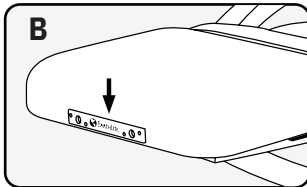

Neck Roll

A neck roll can also be purchased to provide extra comfort for your clients (**Diagram C**).

Foot Rest

A Foot Rest will add 10" to the length of your table.

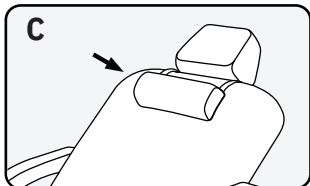


TABLE REMOVAL FROM SHIPPING CRATE

All Ellora table models come completely assembled and ready to use.

CAUTION


If you did not have your table shipped with 'white glove' service, you may be required to remove your table from its shipping crate. We highly recommend that at least two people conduct the following steps to uncrate the table.

Recommended Tools Needed

Work Gloves



Phillips #1 Screw Driver

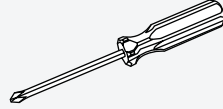


Unpacking your Ellora™ Stationary Table

The following steps will guide you through carefully removing your stationary table from the shipping crate. We recommend wearing work gloves during these steps.


Remove the rectangular cardboard lid from the top of the table. Once you have removed the cardboard lid and surrounding cardboard walls from shipping crate, you will find the wheels and foot glide levelers are seated tightly within wood brackets which have been screwed into the plywood base of the shipping crate.

Step 1

With a Phillips #1 screwdriver (or a powered screwdriver), carefully unscrew and remove the two (2) Front facing wood brackets from the plywood base. **(Diagram A)**

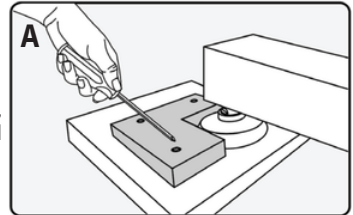


We recommend removing only the two (2) front facing wood brackets **(Diagram A)** that enclose the foot glide levelers. Do not remove the rear brackets which enclose the rear wheels **(Diagram B)**. These brackets are twice as thick as the front brackets, and will be more difficult to remove.

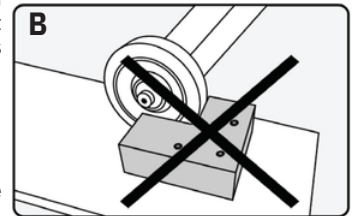

Step 2

Once the front brackets are removed, carefully pull the table straight away from the rear brackets and slide the table onto the floor. We recommend at least two (2) people conduct this step together. **(Diagram C)**

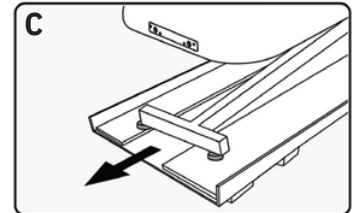


CAUTION

It is possible to lift the table directly out of the shipping crate without removing any of the wood brackets, but this is not recommended because the weight of the table is at least 180 lbs.

Step 3




Once the table is pulled safely away from the shipping crate, make sure the path from the table to your preferred destination is clear of any obstacles or tripping hazards. **(Diagram D)**

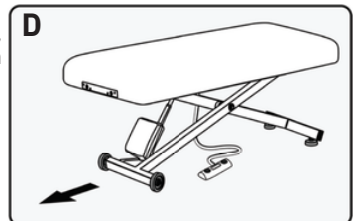

PLACEMENT AND INSTALLATION

Once your table is placed safely on the floor, temporarily plug the table into a power outlet and using the foot control, raise the table to waist height, which will make it easier for moving. Once the table is at the desired height, unplug the power cable from the outlet and place the power cable along with the foot control on top of the table to prevent a tripping hazard. For ease of movement and flexibility, your table was built with two (2) rear wheels and two (2) front foot glide levers.

The following steps will guide you through safely moving your table to the desired location. We recommend that at least two (2) people conduct the following steps:

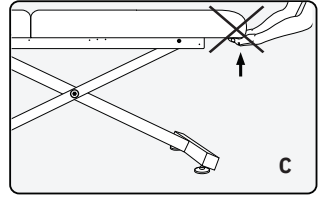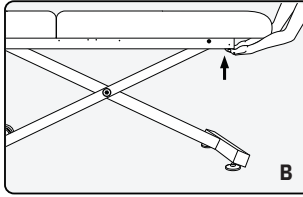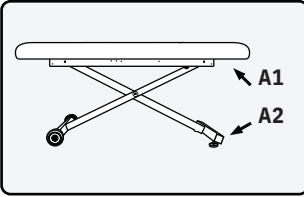

Step 1

Locate the front and rear sections of the table frame underneath the upholstered top as designated by the black arrows above. These are the recommended locations to hold when guiding your table to the desired location (**Diagram A1**).

Note Regarding Step 1:

Locate the front crossbar of the table frame where the foot glide levers are located (**Diagram A2**). This is a secondary recommended section of the frame to hold if you prefer, but may be more difficult. If you choose this location, properly bend at your knees prior to lifting or pulling the table.

Step 2

Ensure there are no obstacles or tripping hazards in the way and carefully pull the table by the frame into the desired location (**Diagram B**). A second person's assistance is recommended to help push the table from the opposite end.



Do NOT pull or push the table from the upholstered top. Moving the table in this fashion can damage the table and put too much pressure on hinges or other components of the table. Damage made to the table by pulling or pushing on the upholstered top may void the warranty (**Diagram C**).




Install this furnishing in treatment area not accessible to the public.

SAFETY ZONE: The treatment area must have 3-ft minimum unobstructed clearance around entire table. Posting a notice is recommended to maintain "Safety Zone Clearance". The designated work area around the table should be clearly marked off.


LEVELING THE TABLE

Not all commercial and residential floors are perfectly level. The Ellora has foot glide levers that swivel into place against the floor. If your table is not already level when you remove it from the shipping crate, follow these easy steps to level your table:

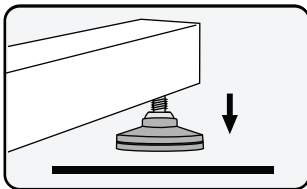


Step 1

Locate which foot glide lever is not even with the floor.

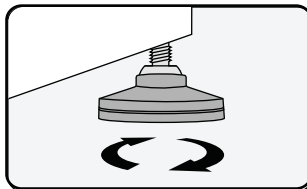

Step 2


Rotate the foot glide lever clockwise or counter clockwise to bring it to ground level.

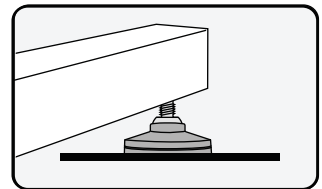


Step 3

Once the foot glide lever is tight with the floor, apply pressure on top of the table to confirm stability.

IMPORTANT SAFETY NOTES

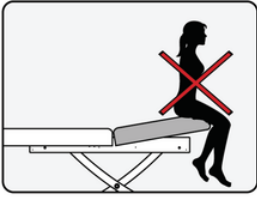

Do not sit or put unnecessary amounts of pressure on the back section of the table when it is in tilt position of any level. This can damage or destroy the tilt mechanism and cause harm or bodily injury to persons. The tilt mechanism is intended solely to be used when a client is properly seated or laying on the table top. Damage caused in this manner may void the warranty.

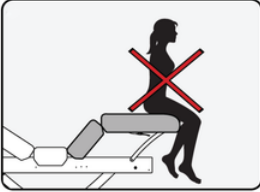

Do not sit or put unnecessary amounts of pressure on the leg section of the table when it is in tilt position of any level. This can damage or destroy the tilt mechanism and cause harm or bodily injury to persons. The tilt mechanism is intended solely to be used when a client is properly seated or laying on the table top. Damage caused in this manner may void the warranty.

SEE BELOW PHOTO FOR PINCH POINT ZONES, and read the following warnings carefully regarding safe use of your table. The Pinch Point Zones must be kept clear at all times.

NOTE: This table does not have pinch point protection, safety shutoff, or crush sensor.




Always use extreme caution when making any position adjustments to your table. Ensure all body parts and objects are clear from all moving parts and pinch points.



DO NOT store or place anything under the backrest or footrest - this can damage the top and/or tilting mechanism and will void your warranty. This area is not designed for storage or as an area to hold accessories and/or massage supplies while performing any type of service.

DO NOT store or place anything between the frame and the floor - the area underneath the table is not designed for storage or as an area to hold accessories, including foot pedal. This can damage the table, frame, and/or accessories, and will void your warranty.

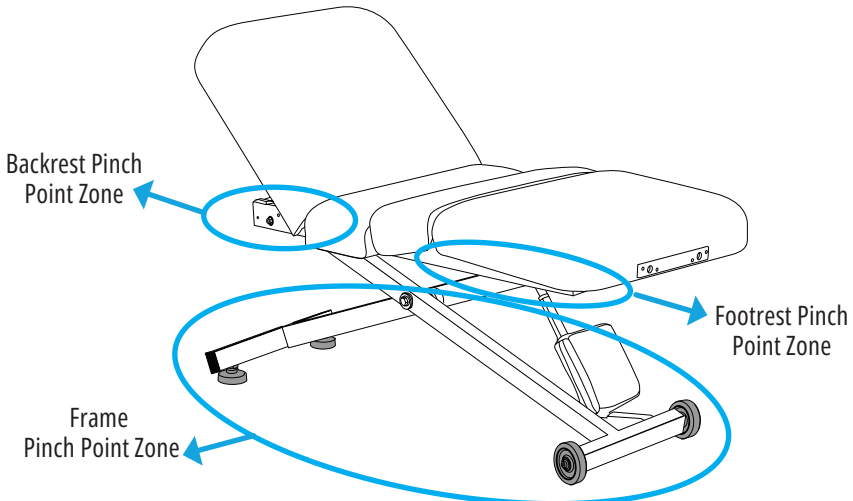

HEIGHT ADJUSTMENT

The Ellora™ comes out of the shipping crate ready to go to work. The foot control is already connected, allowing you to simply raise and lower the table with your feet while your hands remain free to focus on your clients. See below for foot control types and operation.

WARNING

There is no on/off switch on the foot control, so even while the table is not in use, the foot control is still operable. There is a potential for harm or bodily injury when the foot control is being depressed. We recommend unplugging the table from the power outlet when not in use.

Standard Foot Pedal

Lower Entire Table


Raise Entire Table



ETL Option


Lower entire table

Raise entire table

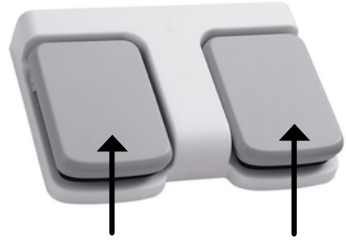


ELECTRIC TILT & SALON TOP

Electric Ellora™ Salon or Tilt top tables are equipped with a hand control. The hand control is already connected and ready for use. See below for explanation of each button's functionality.

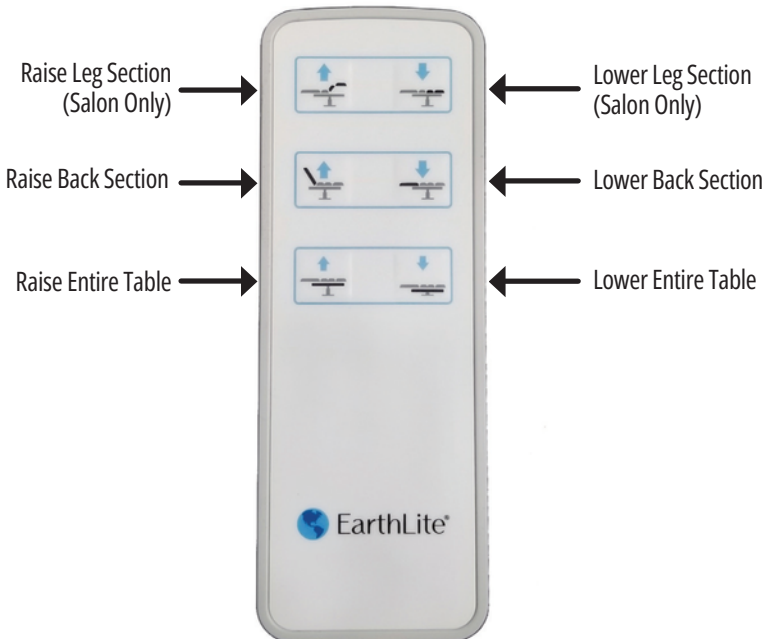


MANUAL TILT TOP

Every Ellora™ with manual tilt functionality is equipped with a locking tilt mechanism which allows you to tilt the back section of the table in 10° increment levels to accommodate your clients. The following steps will guide you through adjusting the back tilt to the desired level.

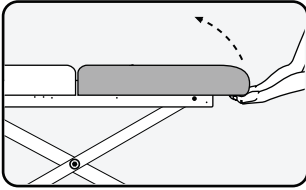


Step 1

To tilt the back section upward from its flat position, carefully place both hands underneath the rear edge of the upholstered top of the tilt section, as pictured above. Lift the tilt section of the table to the desired level. The tilt section will automatically 'click' into position as you lift upward.

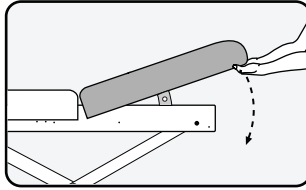

Step 2

To return the tilt section to a flat position, release the locking mechanism by lifting the back section all the way forward past 90°, until the locking mechanism releases.

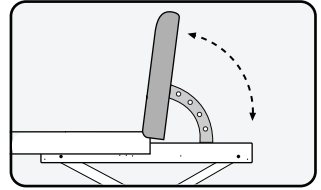


Step 3

Once the locking mechanism is released, lower the tilt section until it is flat (1), then lift again to another desired angle (2).



PNEUMATIC TILT TOP & SALON TOP

For pneumatic Ellora™ tables that come equipped with a non-electric pneumatic assist mechanism, the following steps will guide you through operating the pneumatic controls and adjusting the back and leg sections to the desired level:

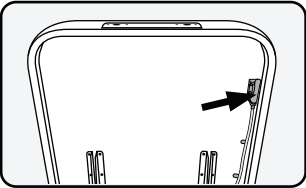


Step 1

Locate the pneumatic handle. Back tilt is located on the right upper side, and leg break is under the leg section of the table.

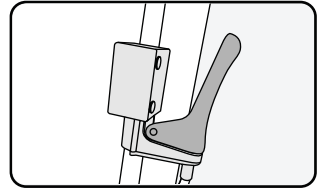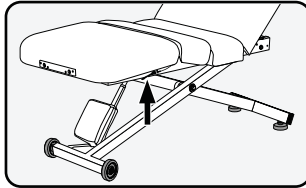


Step 2

The handle is set to the locking position.

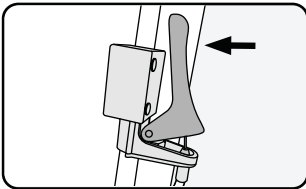


Step 3

Squeeze / press the handle to release the pneumatic assist, which allows you to move the section up or down.

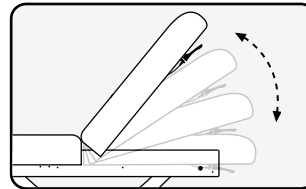

Step 4

With the handle squeezed/pressed, move the back or leg section to the desired level and release the handle. It will lock in place.

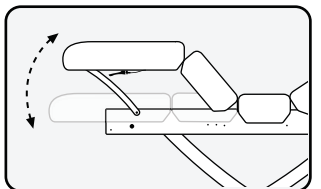


CHEST COMFORT SYSTEM (OPTIONAL)

If your Pro Salon table is equipped with a Chest Comfort System, read below instructions.

CAUTION

Always use caution when making any position adjustments while client is on the table.

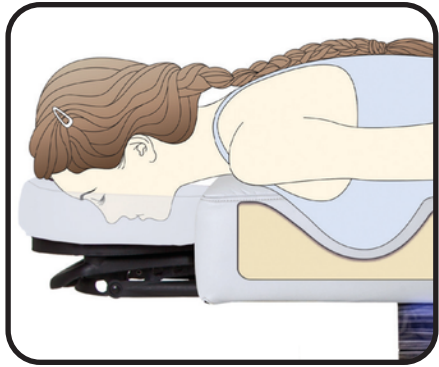

Side View in lowered position

ELLORA LX REPLACEABLE MATTRESS

For the Ellora LX model, you can replace the mattress and cover for extended product life. Follow the instructions below to switch out the mattress and/or cover.



Step 1

Using a Phillips screwdriver, remove the outlet plates from both the head end and foot end of the table.



Step 2

Release cover from the Velcro securing it to the table frame, and carefully remove.

Step 4

Reverse Steps 1, 2, and 3 to place a new mattress and cover, with the final step being to re-attach the outlet plates.



Step 3

Remove mattress if replacing.

STRETCH ASSIST (OPTIONAL)

If your table came equipped with Stretch Assist, see below for instructions on how to set up and utilize the system.



To attach the band to the Stretch Assist bar:

1. Open the anchor strap and lay it flat.
2. Take the velcro end (with velcro facing away from you), and loop the end over the top of the bar.
3. Thread the end of the strap through the buckle and loop back to the other piece of velcro on the strap to attach securely.



Place band over client and buckle the strap.




Pull loose strap end(s) to tighten as needed.



*Shown using optional second strap (Sold separately)


REPLACING A FOOT GLIDE LEVELER

If a foot glide leveler becomes damaged or is in need of replacement, carefully follow these next steps. We recommend that at least two (2) people conduct steps 1 & 5.

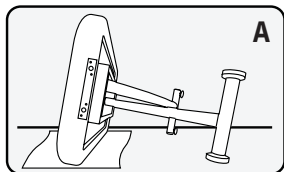


Step 1

Unplug the table and carefully lean the table on its side opposite the foot glide leveler in need of replacement (**Diagram A**).

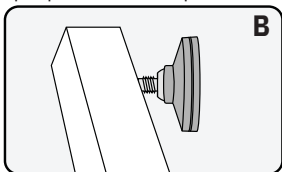


Step 2

Locate the damaged foot glide leveler in need of replacing which is attached to its metal ball joint (**Diagram B**).

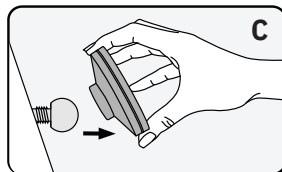


Step 3

Remove the damaged foot glide leveler by pulling it directly off the metal ball joint (**Diagram C**).

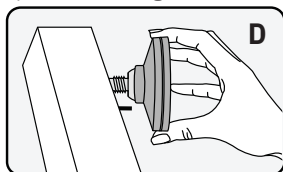

Step 4

Attach the new foot glide leveler to the metal ball joint and push. The Foot glide leveler will snap into place (**Diagram D**).

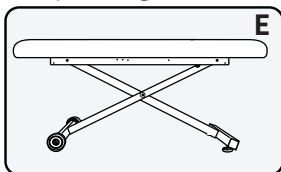

Step 5

Carefully return the table back to its upright position (**Diagram E**).



We recommend at least two (2) people lay the table on its side before conducting step 1 & 5. Lay a towel or blanket down under the table, before resting the table on its side to protect the upholstery.

TROUBLESHOOTING

This helpful troubleshooting guide will provide you with information about how to recognize a problem with your Ellora™, determine possible causes, and find a solution. In the event your specific situation is not covered in this guide, please contact Buy-Rite Beauty at 912-335-4445 so our technical support department can assist you.

Our Customer Service Representative will ask you to provide the following information:

- Your table's serial number (located on the underside of the head section of the table top)
- Description of the problem

Problem	Probable Cause	Solution
My table is wobbling and moving from side to side.	The table is not level with the floor.	Adjust one or both of the foot glide levers until the table becomes level with the floor and is stable.
Table top won't rise or lower. Pneumatic hand lever or finger lever seems stuck	Cables might be caught or tangled.	Check to see that all the cables are free and not caught on any brackets.
Where can I find my table number?	The customer might be looking on the motor.	The table number is located on underside of the table, near head section on a white label. It will read 'SNT' followed by a 6 digit number.
My table is making noise when going up and down.	The center bolt that connects the legs together is too tight.	Remove the plastic cap covering the center bolt and loosen the bolt one half to one full turn.
My table is making a clicking noise and it's not going up or down.	The control box is not relaying the signal from the foot pedal to the actuator.	Replace the control box.
My table is not going up or down and it's not making any sound.	Foot pedal is not relaying signal to the control box.	Replace the foot pedal.

CARE & MAINTENANCE

Perform the following steps as needed for basic maintenance on your Ellora™ massage table and optional accessories. Perform the basic maintenance recommendations for upholstery between client treatments.

Category	Recommendation	Application
Upholstery	Remove day-to-day dirt with a mild soap and warm water solution. For tough stains and to disinfect, use a 15:1 water to bleach solution. Always rinse surface with clean water, and then gently wipe the upholstery. Allow upholstery to air dry before using or storing. Do NOT use Armor All® or citrus-based cleansers. For best results, use only the products suggested above. Alternative cleansing methods may cause serious damage to your table. Such damage would not be covered under the Earthlite warranty program.	<ul style="list-style-type: none"> • Table Bed Surface • Flex-Armrests • Headrest Cushion • Salon Pillow • Bolsters
Steel Table Frame	The powder coated steel frame can be kept clean with a mild detergent and warm water mix. Periodically remove dust from frame with a dry, soft cloth.	<ul style="list-style-type: none"> • Main Steel Frame and Legs
Electronic Components	Anti-static electronic aerosol dusters can be used to keep the surfaces of electronic components, power cables, and plugs free of dust.	<ul style="list-style-type: none"> • Foot Controls • Hand Controls • Motors • Power Cables • Plugs
Lubrication & Mechanical	If necessary, lubricate wheel joints, casters, foot glide levelers, ball joints, and other mechanical parts with a light machine oil. Periodically check table and ensure all bolts remain tightened.	<ul style="list-style-type: none"> • Wheel Joints • Casters • Foot Glide Levelers • Ball Joints, etc.

Common lubrication points

For center bolt and shoulder bolt locations (1 & 2), use of a bearing grease, such as GreaseCo Heavy Duty, is recommended.

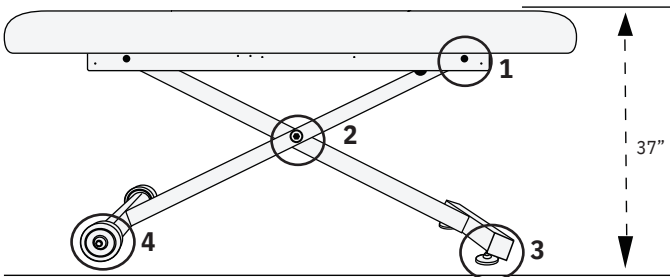


TABLE MODEL COVERAGE

Ellora™ models covered by this user manual.

Salon Top	Tilt Top	Flat Top
Pneumatic Top w/o Accessories 28" table_Code: 162xx 30" table_Code: 148xx 32" table_Code: 149xx	Manual Top 28" table_Code: 151xx 30" table_Code: 153xx 32" table_Code: 155xx	Top 28" table_Code: 150xx 30" table_Code: 152xx 32" table_Code: 154xx
Pneumatic Top w/ Accessories 28" table_Code: 161xx 30" table_Code: 146xx 32" table_Code: 147xx	Pneumatic Top 28" table_Code: 109xx 30" table_Code: 110xx 32" table_Code: 111xx	
Electric w/o Accessories 28" table_Code: 80090xx 30" table_Code: 80091xx 32" table_Code: 80092xx	Electric w/o Accessories 28" table_Code: 80087xx 30" table_Code: 80088xx 32" table_Code: 80089xx	
Salon Accessories 80029xx		

xx = color code for upholstery

LIMITED LIFETIME WARRANTY

Absolute Satisfaction Return Policy

If you are not completely satisfied with your Earthlite portable table, chair, or accessory, you may return it within thirty (30) days for a full refund (excluding shipping & handling charges on the original order). Standard stationary and lift tables may be returned within thirty (30) days for a refund (excluding shipping & handling charges on the original order) less a 20% restocking fee. Custom or upgraded products are non-returnable and non-refundable. Proof of purchase is required to obtain RMA (Return Material Authorization) and all returns must be in like-new condition and in the original packaging. Customers will be responsible for all shipping charges.

Warranty Information

Earthlite offers an industry best warranty against defects in materials, design, and workmanship, to the original owner, as follows:

- Frame - lifetime warranty
- Foam and upholstery covering - warranted for three (3) years
- Lift motors, controls, electronics - warranted for two (2) years from the original date of purchase*
- Accessories and options - warranted for one (1) year from the original date of purchase

This warranty does not cover damage caused by normal wear and tear, misuse, accidents, or neglect, which includes tears in the upholstery and damage caused by cleaners. All warranties are limited to repair or replacement at Earthlite's discretion, shipping not included. Proof of purchase is required to obtain RMA (Return Material Authorization) and customers may be required to return defective items. International customers are responsible for all associated return costs (inbound and outbound), as well as any applicable duties and taxes.

*If within these two (2) years, there is a defect, malfunction, or failure of the lift motor or controls, Earthlite will replace or repair the defective component at our discretion; shipping not included. Parts are warranted for 90 days after replacement or 2 years from the original date of table purchase, whichever comes later.



VISIT OUR SHOWROOMS

Atlanta, GA Showroom
3070 Business Park Drive, Suite H-1
Norcross, GA 30071



CALL US! (912) 758-2659



Join us on
Facebook



Follow us
on Pinterest



Follow us on
Instagram



Connect with us
on Linked In