

F-313A GALVANIC

【F-313A Galvanic Instruction】

By mean of the movements of DC positive and negative ions, the galvanic current treatment is helpful for leading in nutrition and leading out impurity from skin. The nutrition leaded in is to penetrate the water-soluble nutrition into the deep skin, directly supplying nutrition for skin. Impurity leaded out is to import the alkaline solution to soften the aged horn, decompose the grease balance the skin secretion, leading out the impurity and metal ions at the same time; The electronic ions of positive electrode may lead the acidic solution (shrinking lotion) into the skin, and harden the skin tissue, comfort nerve and diminish blood supply as well. It is mainly used after beauty treatment to alleviate skin being red and swollen and to avoid inflammation. The electronic ions of negative electrode may lead alkaline solution into skin to soften skin tissue, irritate nerve and increase blood supply. Mainly used in aged and greasy skin.

【Galvanic Operation Instruction】

1. Connecting the accessories corresponding to the instrument before turning on the power. Client grasps the metal pole, and beautician holds the handle of the roller.

2. Changing the electrode of the roller by adjusting leading in (+R) and leading out (-G). The treated part must be clean and keep it clean and dry before doing the nutrition leading in. Turn the electrode button to (+R), now the roller is positive while the metal pole is negative. Beautician spreads some liquid nourishing lotion evenly on the treated part, then cover the roller with a piece of cotton or sponge that has dipped into the acidic lotion (like shrink lotion) or directly cover the roller with a piece of cotton with nourishing lotion before doing this treatment. While doing leading out treatment, should change the electrode button to (-G), now the roller is negative while the metal pole is positive. Cover the roller with a piece of cotton or sponge that has dipped into the alkaline lotion (like detergent lotion) before doing this treatment. It is to balance and decompose the grease of skin.

2. Power on "IONOS", select the operation form: leading in "+R" or leading out "-G", turn "I/Energy" to the minimum. Touch the treated part with the roller, slowly adjust "I/Energy" to appropriate intensity which client feels mildly stung. Smoothly slide the roller, and the cotton must keep moisture all the way for better effect.

【Panel Instruction】

①POWER

②Indicator for

galvanic polarity

③Polarity

④TIMER

⑤ENERGY“+”button

⑥TIMER”+”button


⑦ON/OFF

⑧ENERGY

⑨Switch button of galvanic probe

⑩ENERGY“-”button

⑪TIMER”-”button

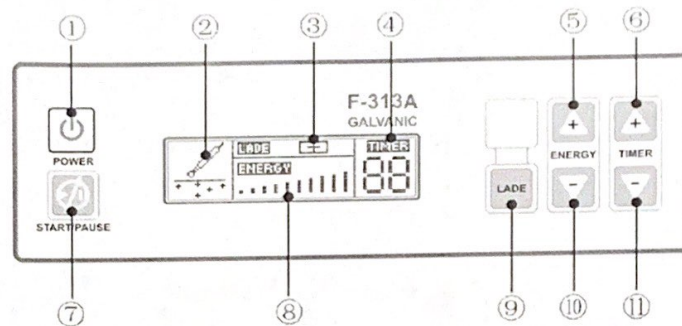

【Cautions】

1. After treatment, the roller and metal pole must be clean and keep in dry to avoid damage.
2. When the roller is negative, the stung feeling will be felt more intensive than it is positive, otherwise, it will be felt weaker.
3. Before doing the treatment, clients will be required to take off jewels, it

is not applicable for someone who there are any metal objects inside.

4. Pregnant women and people who have sensitive, injured, inflamed skin or blood capillary expansions are not allowed for the treatment.

5. The lotion with leading in or leading out must be used in accordance with the roller (negative or positive). While the roller is positive, the lotion should be acidic (+), while the roller is negative, the lotion should be alkaline (-). Please use the PH paper for verification if the lotion is not indicated.

【Warning】

1. With the following diseases, not suitable for the use of this instrument, such as heart disease, infectious disease patients, allergic patients and the patients who have metal in body during therapy period.

2. Don't let the leading-out probe and leading-in probe touch together for a long time when they are working.

3. When the lotion of the skin surface cannot moisten skin, it should be added in time.

4. Prohibit to use the sliding wheel which can not rotate, this accessory

must be changed

5. Please pay attention to the connecting wire between the machine and accessory when working, don't over-pull it.

6. Do not let water or other corrosive liquid splash to the machine when using.

7. In order to avoid scratching skin, prohibit to use the bead probe and roll probe which the surface of stainless steel is defected、rusted or polished. The export-in and export-out probe should be protected carefully, don't let other hard objects scratch the probes.

8. In the course of the host if there are unusual circumstances, please immediately shut down and unplug the power plug, please contacts the dealer near you or invites the professional maintenance for inspection. Prohibit opening the chassis for maintenance unauthorized

9. If the powers cord damage, we must use a dedicated line or buy the special component from the manufacturer or designated maintenance point to replace.