

F-316A Vacuum & Spray Instrument

【F-316A Vacuum & Spray Functions】

If black-head, white-head and acne grow in the face, they not only influence the appearance, but also block the pores. When they become worse, they will block the blood circulation, and influence the skin's metabolism. After the influence cumulates for a long period, the pores will become bigger and bigger, and the skin will become rougher and rougher. This instrument can suck the black-head, white-head, acnes, blains and clean the besmirch and oil in the pores effectively. It can also spray the fog of astringent, diminish inflammation water and shrink water. It can shrink the pores effectively and has the functions of diminish inflammation, easing pain and supplement nutrition.

【Usage Method】

1. Black-head vacuum: Connect the single air pipe to the corresponding vacuum pipe marks nozzle, the other head to the small glass vacuum pipe. Three small glass vacuum pipes for option to perform black-head vacuuming.

Turn on the power of "VAC-SPRAY", and adjust the proper energy and time. Then press the "START/PAUSE" to start work.

It is advisable to stuff some absorbent cotton into the glass vacuum pipe for stopping dirt going into the pipe so as to prevent damaging the

instrument. There's a small hole in the front end of the vacuum pipe, firstly stop the hole by finger as the illustration, when the vacuum pipe touches the face the finger can move away or just move partly to control the vacuum force.

The glass vacuum pipe must not stay on the face too long so that it will scald the face. It is normal if there is no vacuum at the beginning, it will resume to normal condition after a while.

2. Spray: Connect one head of the single air pipe to the corresponding spray bottle marks nozzle, the other head to the small spray bottle.

Turn on the power of "VAC-SPRAY", press the "START/PAUSE" key to start work, and adjust the energy and time properly.

Two spray bottles that can contain different liquid, it will spray out by pressing the nozzle on top of the spray bottle with finger, spray downward from forehead during working, prevent liquid going into the nose, ask client to close eyes and shut mouth. Keep a adequate distance from the face.


【Illustration of panel】

①Power

②Vacuum & Spray indicator

③Energy indicator

④Time indicator

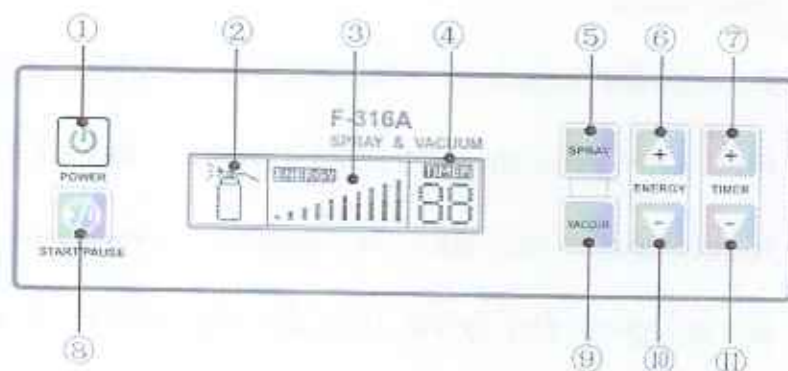

⑤ Spray working key

⑥ Energy increase key

⑦ Time increase key

⑧ Start/Pause

⑨ Vacuum working key

⑩ Energy decrease key

⑪ Time decrease key

【Notes】

1. Before using the function of vacuum, unscrew the glass vacuum pipe to check if there's absorbent cotton. The absorbent cotton can stop dirt going into the pipe so as to prevent damaging the instrument.
2. The absorbent cotton in the glass vacuum pipe must be checked or replaced before usage each time. The glass vacuum pipe will be blocked if there is too much dirt accumulates on the absorbent cotton.
3. After working, all the air pipes and accessories should be cleaned, sterilized and keep in dry place.

【Warning】

- 1、 It is forbidden to insert the Spraying tube into the Vacuum sticker collector, in order to prevent the dressing water is sucked into the pump to damage the machine.

2. When using the spray function, it is forbidden to plug up the output connector of "Spray" by anything or fold the silica gel pipe, this will damage the host.
3. The broken or cracked glass vacuum pipe is strictly prohibited to use.
4. Don't pull the silica gel pipe roughly, or it will damage the pipe.

