

F-318A Cold & Hot Hammer


Brief introduction

1. With the principle of cold & heat conversion, the product uses special refrigeration film, could convert energy to the temperature you need in short time.
2. Can activate cell, improve and purify skin quality; smooth wrinkles and crow's feet more naturally and thoroughly.
3. The alternate use of cold and heat, could expand and shrink pore fastly, give them movement of expansion and shrinkage, thus give the skin vitality and vigor for the elastic.

Operating Instruction

1. Connect host engine and accessories lines well.
2. After press the power switch, first set cold therapy or hot compress function; temperature and time again; and then press the "START/PAUSE" to start the host; last, open the switch on the handle of the hammer.
3. For first use, it takes about 5 minutes to work normally after complete the setting.
4. When one treatment is finished, 3 minutes is needed to switch to another.

Panel Instruction

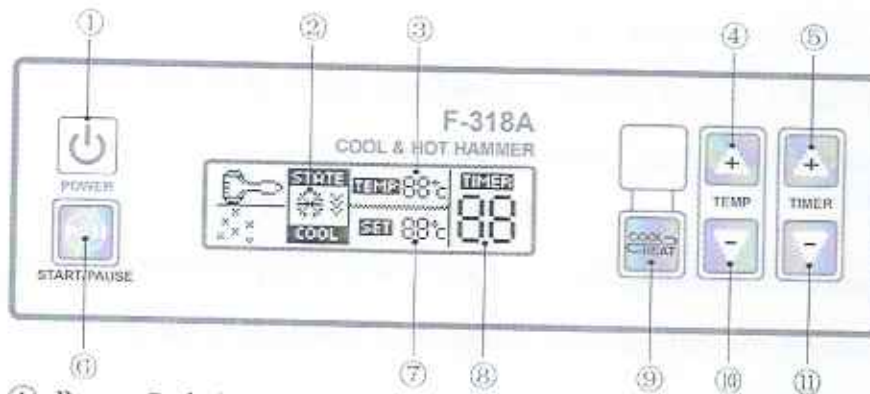

- ① Power Switch
- ② Cold/Hot Indicator
- ③ Real-time Temperature Indicator
- ④ Temperature Increasing Button
- ⑤ Time Increasing Button
- ⑥ Start/Pause Button
- ⑦ Setting Temperature Indicator
- ⑧ Time indicator

- ⑨ Cold/Hot Switch Button
- ⑩ Temperature Decreasing Button
- ⑪ Time Decreasing Button

Cautions:

1. Indoor use only, because the working condition of this instrument would change as the environment temperature changes. The instrument probably can't work well if the temperature is too high or too low, the best is to operate in the room with the temperature about 25°C
2. The scattered gas tank shouldn't be blocked. if so, some condensed water may produced inside the hammer, thus may cause short-circuit fault.
3. When the host engine shows in the state of working after starting the "START/PAUSE", but the cold & hot hammer still don't work, pleas check whether the switch of the hammer has been adjusted to "--" or not.
4. Besides the time of primary seeting starting time and switching cold/hot function, you also need to add the time of temperature going up/down (start time: 5 minutes; switch cold/heat: 3 minutes).

Warnings:

1. The two ends of the cold & hot hammer are made of aluminum alloy, they have soft hardness, so couldn't be used as hitting tools.
2. Scratches arise on both ends, new accessories need to be replaced for next use.
3. Don't pull the spring line too much during the process of using, pay attention to the operating range.
4. The host engine and cold & hot hammer shouldn't be tear apart privately; If there are any unnormal, contact the distributor for maintanance please; open the case to repair personally is prohibited.
5. When arise the following cases: the powerline or plug of the host engine/ cold & hot hammer in the attachment is damaged, tured over, damaged or get wet by the water, don't continue to use the machine again. Please send the machine back to the service centre for check and maintain.
6. Using in damp environment is banned, such as bathroom, etc.